THERALINE

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Kirschkerneissen

WÄRME- & KÄLTEKISSEN

DE KIRSCHKERNKISSEN

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EN CHERRY STONE PILLOW

Use for warm:

Helps relief tension in neck and other back areas. Can support heating of stomach an digestion irritations, so as musclepain, rheumatic pain and lumbago. Helps relief menstruation pain.

Use for cold:

Helps relief headache, lumbago, muscle tension, bruises, joint infections.

For warming in an oven place the Cherrystone insert in the oven on a baking pan for 10 minutes at 150° C. For cooling effects simply place in freezer for up to one hour. Do not leave for an extended period of time. For warming by microwave first dampen the insert by sprinkling water on it. This will prevent the cherrystones from drying out, especially with frequent use. This is very important. Heat for 45 seconds on medium power. If necessary, continue heating in 15 second increments only. After you have used the insert and you wish to reheat it, **do not** reheat it for any longer than a total of 30 seconds.

Attention: Fire hazard when overheating!

Caution: Even slightly overheating will cause an unpleasant burned smell which is permanent! To enjoy long-term use, please follow heating instructions carefully. Also, to prevent burns, make sure to test the temperature on the inside of your wrist for 3 seconds before use.

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